



SPECIAL FEATURES

- IP68 rated medical keyboard
- Adjustable brightness of backlight (TOUCH+)
- Fully washable and disinfectable
- 109 keys incl. num pad and track pad
- Waterproof silicone USB protection cap
- Excellent tactile feeling
- Integrated Clean Mode for cleaning
- Magnetic mounting
- EN60601-1-2 Ed 4 certified

SPECIFICATION

| | | | |
|-----------------------|---|------------------------|--|
| Number of keys: | 109 (US-Layout) | Interfaces: | USB 2.0 |
| Keyswitch technology: | carbon contact | Cable length: | 15cm |
| Switching cycles: | approx. 2 mill. (per key) | Operating temperature: | 0°C ~ +55°C |
| Protection level: | IP68 | Storage temperature: | -30°C ~ +70°C |
| Housing material: | silicone | Dimensions: | 365(L) x 134(W) x 14.2(T) mm |
| Housing color: | white & black | Weight: | approx. 615g |
| Backlight: | blue color, 5 level adjustable brightness | Certification: | EN60601-1-2 Ed 4, CE, FCC, RoHS, REACH |
| | | Warranty: | 2 years |

ORDER INFORMATION

| Cat. No. | Product description | Colour | Layout |
|------------|-------------------------|--------|------------------|
| KSI-U10210 | CLEANTYPE® PRIME TOUCH+ | White | US/DE/JP/KR/THAI |
| KSI-U10211 | CLEANTYPE® PRIME TOUCH | White | US/DE/JP/KR/THAI |
| KSI-U20210 | CLEANTYPE® PRIME TOUCH+ | Black | US/DE/JP/KR/THAI |
| KSI-U20211 | CLEANTYPE® PRIME TOUCH | Black | US/DE/JP/KR/THAI |

Other layouts, configurations and interfaces on request.
Customized key function on request (Firmware update via USB)

EN60601-1-2 Ed 4 certified

EN 60601-1-2 Ed 4 applies to the basic safety and essential performance of Medical Equipment(ME) equipment and ME systems in the presence of electromagnetic disturbances and to electromagnetic disturbances emitted by ME equipment and ME systems.

SHORT OPERATING INSTRUCTIONS



Press and hold lock key for 5s to enable Clean Mode. Clean Mode is indicated by breathing backlight.



Adjust backlight level with this button. Five level can be adjusted including off.

CLEANING INSTRUCTION

For further details please refer to below link:
<http://www.gett-group.com/content/service-cleaning-instructions>